EN

COSMEDENT www.cosmedent.com | 800.621.6729
Enamelize® | A COMPOSITE POLISHING PASTE

INSTRUCTIONS FOR USE

DESCRIPTION

Enamelize® The final polish! **Enamelize** will bring natural tooth structure, dental composite resin, porcelain and metals to the highest polish attainable. This dental polishing paste will greatly increase the gloss, luster and surface smoothness of your restoration. Use **Enamelize** as the final polish on microfills, nanofills, microhybrids, metals and natural dentition. It is also the best material for the routine care of bonded restorations. **Enamelize** creates a shine that is so beautiful you must try it to believe it.

COMPOSITION

Enamelize consists of water, aluminum oxide, emulsifier, glycerin, vegetable oil and methylparabene.

INDICATIONS FOR USE

- Polishing of composite materials.
- Polishing of all metal restorations.
- After diamond polishing paste on porcelain.
- After use of prophylactic paste on natural dentition.

DISPOSAL INSTRUCTIONS

Dispose of contents/ container in accordance with local regulation.

CONTRAINDICATIONS

Enamelize is contraindicated if patient is known to be allergic to any of the ingredients of **Enamelize**.

SIDE EFFECTS

May cause irritation in sensitive patients.

INTERACTIONS

None

DIRECTIONS FOR USE

- ① Apply **Enamelize** to areas you wish to polish.
- ② Light pressure buffing with a rubber prophyl cup or felt wheel is all that is needed to finish with a sheen that will amaze you and will enhance the quality of your restoration.
- ③ A soft buff with a dry piece of gauze will increase the luster.

HYGIENE/DISINFECTION

Disinfection of the syringe can be carried out using commercially available disinfecting solution.

NOTICE

Serious incidents must be reported to DeltaMed GmbH, the responsible importer and the responsible authority

WARNING

Enamelize may cause irritation in sensitive persons. In case of contact with the eyes, rinse immediately with plenty of water and seek medical attention.

STORAGE

- Store at 39-75°F (4 - 24°C).
- Shelf life: see information on syringes and/or packages.
- Keep away from sunlight.
- Keep material out of reach of children.
- Do not use after the expiration date.

DISPOSAL

Do not allow to enter into surface water or drains. Dispose of waste according to applicable legislation.

PACKAGING

Enamelize 3 gram individual syringe, **Enamelize** 36 gram tube

Rx IMPORTANT
PRODUCT
INFORMATION

Federal (U.S.) law restricts this device to sale by or on the order of a dentist.

Note: This product was developed for use in the dental sector and should be used in accordance with the instructions for use. The manufacturer does not accept liability for damage caused by its use for any other purpose. The user is also obliged to accept responsibility for checking before use that the material is suitable and can be used for the intended purpose, particularly if this is not listed in the instructions for use.

Canada Site Licence Holder:
CLINICAL RESEARCH DENTAL
167 Central Ave, Suite 200
London, Ontario, Canada N6A 1M6
Establishment License No. 666

Australia Sponsor:
ERSKINE ORAL CARE
5-9 Kylie St,
Macksville NSW 2447
Australia

Australia Sponsor:
Amalgadent Dental Supplies Pty Ltd
334 Glen Huntly Road
Elsterwick, Victoria 3185 Australia

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Cosmedent® Inc.